coldes Textilien GmbH

**Heinweis:**

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**Note:**

Masks should NOT be worn by children under the age of 2 or anyone who has trouble breathing, is unconscious, incapacitated, or otherwise unable to remove the mask without assistance.

This mask must be immediately changed when it is damaged or wet.

This mask is just for single use.

Do not reuse the mask. We recommend the maximum application time of 8 hours not to be exceeded. Do not wash, disinfect, or heat the mask.

People who are allergic to polypropylene must not use this mask.

**Safety Equipment**

- Control the mask before get in the using area.
- In working area there must be at least %19,5 oxygen otherwise mask not able to use.
- The masks do not protect you from gases and steam
- Change the mask if there will be any obvious and realizable extra air resisting.
- If there is any leakage redo the 1-2-3-4 process again.

Notifizierter Körper/Notified Body  
Universal Sertifikasyon ve Gözetim Hizmetleri Tic.Ltd.Şti. NB2163

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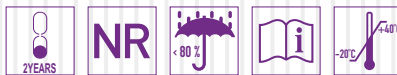
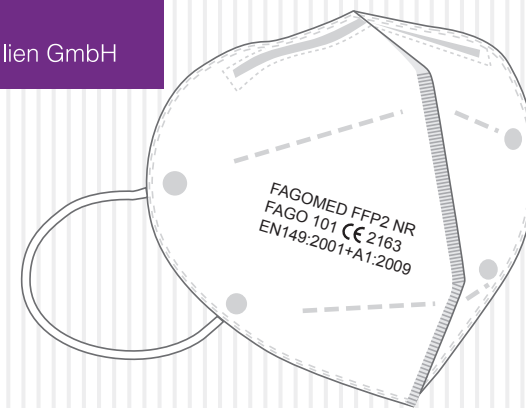
- More Comfortable to Wear
- Low Respiratory Resistance
- High Filtration Efficiency

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**USER INSTRUCTIONS**

- Do not touch**  
Do not touch the mask outside in the danger area. In this case, immediately put on a new mask.
- Change your mask**  
In case of feeling wetness and humidity either inside or outside of your mask, please put on a new mask immediately.
- Maximum wearing time**  
The maximum time that you can wear your mask, for one time is 8 hours. Please do not exceed maximum using time.
- Taking the mask off**  
Take off the mask only by pulling down the ear straps.

Turkey  
Discover  
the potential

CE 2163