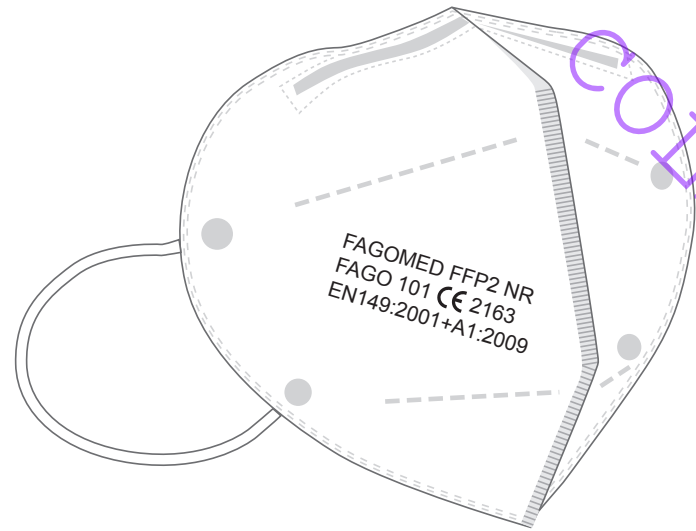
coldes Textilien GmbH

FAGOMED FFP2 NR  
FAGO 101 CE 2163  
EN149:2001+A1:2009

Automatisch absorbierender Filtertyp Anti-Fog / Automatic Absorbing Filter Type Anti-Fog

Verbesserte Atemschutzmaske / Enhanced Respirator

# SCHUTZMASKE PROTECTIVE MASK

FFP2 NR  
EN149:2001+A1:2009

MASKE / MASK / MASQUE / MASKEER  
MASCHERE / MASCARA / MASCARAR

CE  
2163

1  
STCK  
PCS

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- More Comfortable to Wear
- Low Respiratory Resistance
- High Filtration Efficiency

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## HOW TO WEAR YOUR MASK

	<b>Place your mask</b> Place the mask with the inner surface facing up. Place it on your chin and over your nose.	1
	<b>Put on ear band</b> Put the ear bands right & left over both ears at your own discretion	2
	<b>Adjust the nose clip</b> Use both hands to carefully adjust the nose piece to your nose.	3
	<b>Check fit</b> Make sure the mask completely covers your nose, mouth & chin & fits snugly.	4

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## USER INSTRUCTIONS

1	<b>Do not touch</b> Do not touch the mask outside in the danger area. In this case, immediately put on a new mask	
2	<b>Change your mask</b> In case of feeling wetness and humidity either inside or outside of your mask, please put on a new mask immediately.	
3	<b>Maximum wearing time</b> The maximum time that you can wear your mask for one time is 8 hours. Please do not exceed maximum using time.	
4	<b>Taking the mask off</b> Take off the mask only by pulling down the ear straps.	

ISO 9001 :2015 (QMS)  
ISO 10002 :2018  
ISO 13485 :2016  
ISO 14001 :2015 (EMS)  
ISO 45001 :2018 (OHSMS)

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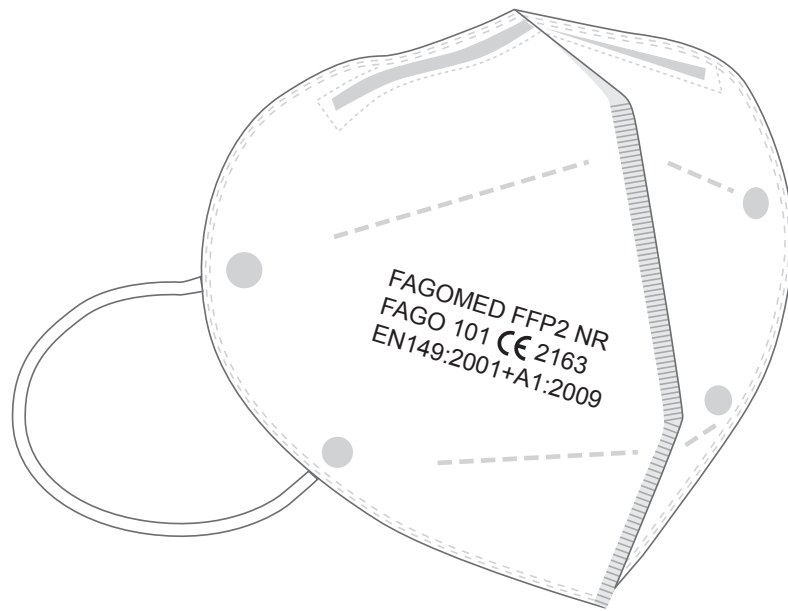
### HOW TO WEAR YOUR MASK

	<b>Place your mask</b> Place the mask with the inner surface facing up. Place it on your chin and over your nose.	<b>1</b>
	<b>Put on ear band</b> Put the ear bands right & left over both ears at your own discretion	<b>2</b>
	<b>Adjust the nose clip</b> Use both hands to carefully adjust the nose piece to your nose.	<b>3</b>
	<b>Check fit</b> Make sure the mask completely covers your nose, mouth & chin & fits snugly.	<b>4</b>

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### USER INSTRUCTIONS

<b>1</b>	<b>Do not touch</b> Do not touch the mask outside in the danger area. In this case, immediately put on a new mask	
<b>2</b>	<b>Change your mask</b> In case of feeling wetness and humidity either inside or outside of your mask, please put on a new mask immediately.	
<b>3</b>	<b>Maximum wearing time</b> The maximum time that you can wear your mask for one time is 8 hours. Please do not exceed maximum using time.	
<b>4</b>	<b>Taking the mask off</b> Take off the mask only by pulling down the ear straps.	

ISO 9001 - 2015 (QMS)  
ISO 10002 - 2018  
ISO 13485 - 2016  
ISO 14001 - 2015 (EMS)  
ISO 45001 - 2018 (OHSMS)

LOT  
F-003-2021-01-1-1

2 YEARS

NR

Model No  
FAGO101

**Manufacturer** FAGO MEDİKAL San. ve Tic. Ltd. Şti.  
15 Temmuz Mh. Cami Yolu Cd.  
No: 106/Z1 Bağcılar - İstanbul - Türkiye  
Phone: (+90-212) 630 67 55 (+90-212) 630 67 56  
www.famexmask.com  
e-mail: info@fagomedikal.com  
**Deutschland Distributor**  
COLDES Textilien GmbH  
Am Alten Fliess 66-50129 Bergheim Deutschland  
www.coldes-medical.com

**1** PCS  
STCK

Turkey Discover the potential  
CE 2163  
ISO 9001 ISO 13485 ISO 14001 ISO 45001